

# *Navigator*<sup>™</sup>

Ureteral Access Sheath Set


Boston  
Scientific

For easier,  
less traumatic scope  
manipulation  
during ureteroscopy



# The Navigator™ Ureteral Access Sheath Set

is designed to provide easier, less traumatic scope manipulation during ureteroscopy.

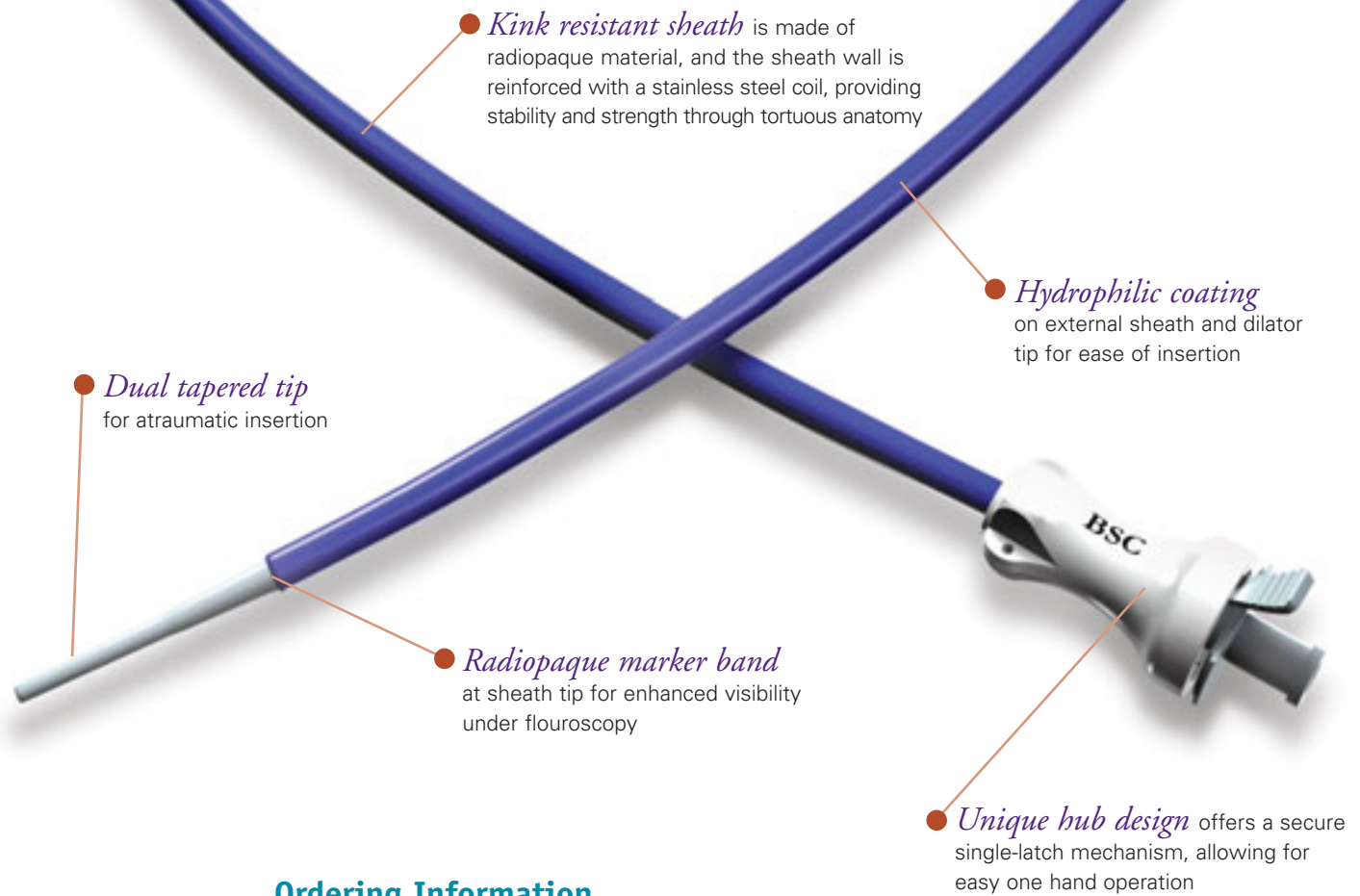

## Ordering Information

Product Code	Size	Packaged
M0062502000	11/13Fr x 28 cm	Unit
M0062502011	11/13Fr x 28 cm	5-Pack
M0062502020	11/13Fr x 36 cm	Unit
M0062502031	11/13Fr x 36 cm	5-Pack
M0062502040	11/13Fr x 46 cm	Unit
M0062502051	11/13Fr x 46 cm	5-Pack
M0062502060	13/15Fr x 28 cm	Unit
M0062502071	13/15Fr x 28 cm	5-Pack
M0062502080	13/15Fr x 36 cm	Unit
M0062502091	13/15Fr x 36 cm	5-Pack
M0062502100	13/15Fr x 46 cm	Unit
M0062502111	13/15Fr x 46 cm	5-Pack

**Indication for Use:** The Navigator™ Ureteral Access Sheath is indicated for use in endoscopic procedures to facilitate the passage of endoscopes, urological instruments and for the injection of fluids into the urinary tract. **Contraindications:** Patients who are contraindicated for Retrograde urologic procedures; Presence of tight strictures which would limit use of the device; Presence of large obstructing distal ureteral stone. **Potential Complications:** Potential complications associated with the use of the transurethral access device include, but is not limited to: Mucosal irritation, inflammation and edema; Urethral strictures; Acute bleeding or hemorrhage; Urethral, bladder, or ureteral perforation; Other injury to the urinary tract. **Precautions:** To minimize resistance during advancement, ensure the hydrophilic coating on the dilator and sheath is activated with saline or sterile water prior to placement. DO NOT bend or kink the catheter prior to placement, to do so could damage the integrity of the catheter and result in patient injury. If resistance is encountered during advancement or withdrawal of the catheter, STOP. DO NOT continue without first determining the cause of the resistance and taking remedial action. DO NOT advance the sheath without the dilator in place, to do so could cause injury to the patient. **Caution: Federal law (U.S.A.) restricts this device to sale, distribution and use by or on the order of a physician.** Refer to the Directions for Use provided with this product for complete instructions, warnings, and precautions prior to using this product.

**Boston  
Scientific**

*Delivering what's next.™*

Boston Scientific Corporation  
One Boston Scientific Place  
Natick, MA 01760-1537  
www.bostonscientific.com/urology

**Ordering Information**  
888.272.1001

Copyright © 2006 by  
Boston Scientific Corporation  
or its affiliates. All rights reserved.

BVU1080 RevB 5M 1/06 - 1/08