

Trapezoid™ RX

Wireguided Retrieval Basket

**Boston
Scientific**
Advancing science for life™



RX Biliary System™

We're in!™

Trapezoid™ RX

Wireguided Retrieval Basket

The Trapezoid RX Wireguided Retrieval Basket



is designed for the crushing and removal of stones in the biliary duct. The Flexi-Stiff™ Catheter construction provides a flexible and sturdy catheter designed to bend with biliary anatomy while maintaining pushability. The metal alloy basket is designed to maintain basket symmetry even after difficult stone manipulation.

RX Biliary System™

We're in!™

The RX Biliary System helps put the physician in control of the guidewire. Using the RX Locking Device, the physician can secure the guidewire to maintain access while exchanging devices.

The RX Biliary System and its compatible devices offer a broad portfolio of products for pancreatico-biliary stone and stricture management.

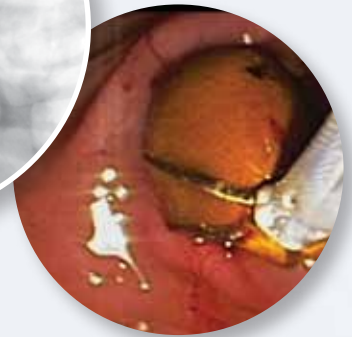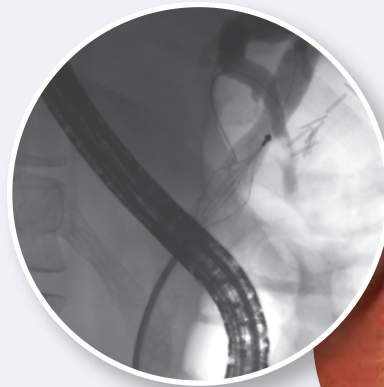


Tip Disengages

Tip disengages to facilitate basket removal if stone does not break, helping to reduce risk of basket impaction.



Mechanical Stone Crushing Option


The Alliance™ II Inflation System can be used to crush large stones trapped within the basket with scope in place

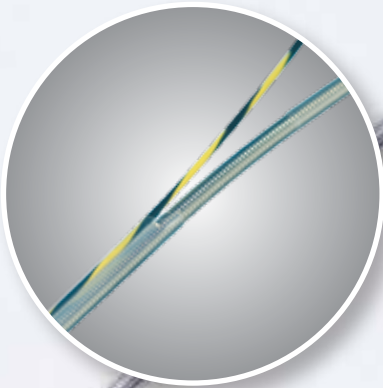

Maintains Guidewire Access

Distal side-port guidewire channel facilitates duct access

- Side-port designed to lead basket past stones and help retrieve stones
- Distal side-port channel accepts .035" short 260cm guidewire or 450cm traditional guidewire to accommodate physician technique preference
- Injection with wire in place accommodates cholangiography without device exchange
- The RX Locking Device facilitates multiple sweeps of the duct during stone retrieval by maintaining guidewire position
- Wire can be locked and unlocked at any time during the procedure

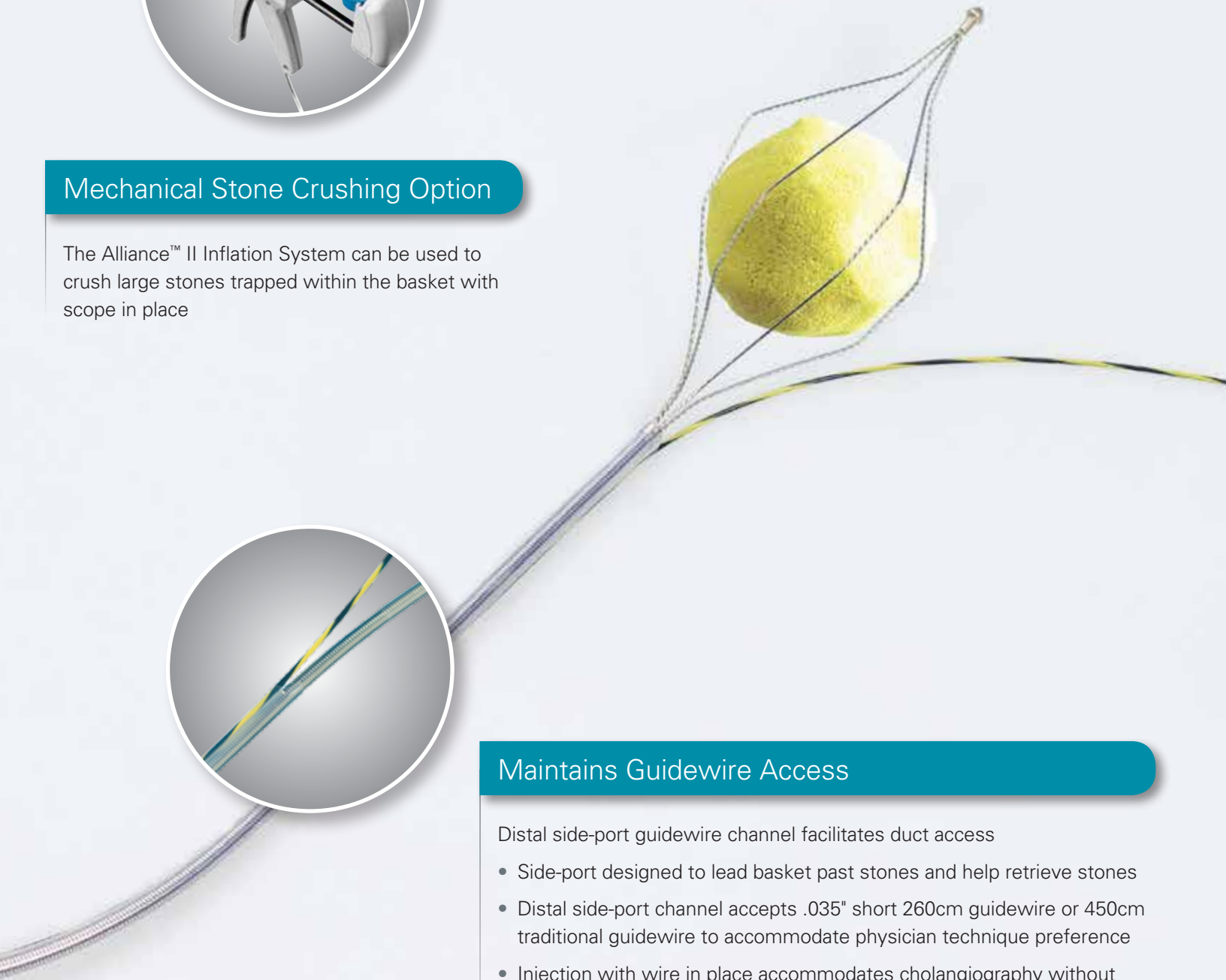

Trapezoid™ RX

Wireguided Retrieval Basket



Trapezoid RX Wireguided Retrieval Basket

Order Number	GTIN	Description	Open Basket Diameter (cm)	Required Working Channel (mm)	Recommended Guidewire (in)	Packaging
M00510860	08714729296379	Trapezoid RX Wireguided Retrieval Basket	1.5	3.2	.035	Each
M00510870	08714729296386	Trapezoid RX Wireguided Retrieval Basket	2.0	3.2	.035	Each
M00510880	08714729296393	Trapezoid RX Wireguided Retrieval Basket	2.5	3.2	.035	Each
M00510890	08714729296409	Trapezoid RX Wireguided Retrieval Basket	3.0	3.2	.035	Each

*Recommended guidewire: .035" Hydra Jagwire™ Guidewire

Alliance™ II Inflation System

Order Number	GTIN	Description	Packaging
M00550620	08714729283881	Inflation Handle	Each
M00550601	08714729129332	Single-Use Syringe / Gauge Assembly	Box 5

Boston Scientific

Advancing science for life™

Boston Scientific Corporation
300 Boston Scientific Way
Marlborough, MA 01752-1234
www.bostonscientific.com

Ordering Information
1.888.272.1001

©2016 Boston Scientific Corporation
or its affiliates. All rights reserved.

ENDO-333814-AA May 2016

All trademarks are the property of their respective owners.

Indications, Contraindications, Warnings and Instructions for Use can be found in the product labeling supplied with each device.

Caution: Federal (USA) law restricts this device to sale by or on the order of a physician.